EN Instructions for use

Salvequick Foot Rescue

Intended use: Treatment of foot fungus and simultaneous treatment of heel cracks (rhagades), calluses (hyperkeratosis) and dry feet.

1. How does Salvequick Foot Rescue work?

Salvequick Foot Rescue is a patented medical device class IIa that has water-binding and antimicrobial properties.

2. Ingredients

Aqua, 1,5-Pentanediol, Cetyl Alcohol, Caprylic/ Capric Triglyceride, Glycerin, Isostearyl Isostearate, Dimethicone, Glyceryl Stearate, PEG-75 Stearate, Ceteth-20, Steareth-20, Xanthan Gum, Artemisia Abrotanum Leaf/Stem Extract, Disodium Phosphate, Sodium Phosphate.

3. Use

Salvequick Foot Rescue can be used to treat and prevent the spread and / or aggravation of foot fungus, heel cracks, calluses and dry feet. To be used by adults. Apply 1 pump on each foot. The volume depends on the size of the treatment area. Apply to clean and dry feet so that the entire area to be treated is covered. Treat both feet to prevent spreading. Wash hands after application to avoid contamination of other areas.

4. Treatment

Apply twice a day until all symptoms are completely eliminated. The treatment time is individual and depends on the initial severity of the symptoms. Initial effect on symptoms is usually seen after two weeks of treatment. For treatment of foot fungus an eight-week treatment period is recommended. It is important not to finish the treatment too early to prevent recurrence. If no improvement can be seen after 4 weeks, consider alternative treatment and/or contact a medical professional for advice. If you are a diabetic, suffer from reduced blood circulation or recurrent and / or long-term problems with your feet contact a medical professional for advice.

5. Description of the container

Salvequick Foot Rescue is packed in an airtight package, a long shelf life is obtained when the product is protected from the environment. The bottle delivers about 1 ml per pump.

6. Warning and precautions

- To be used by adults. Salvequick Foot Rescue is intended only for use on the feet.
- Do not use on open wounds or on other parts of the body. Do not use the product if the package is damaged or if the expiry date stated on the package has passed. In case of accidental ingestion, a health care professional should be consulted, and this leaflet provided. Do not use if you are allergic or hypersensitive to any of the ingredients included or if you suffer from glutaric acid or kidney insufficiency. Do not combine Salvequick Foot Rescue with other products intended for foot fungus without first consulting a health care professional. Discontinue use of the product if irritation or allergic reaction occurs.
- In case of any side effects or any abnormal reaction, a health care

professional should immediately be consulted, and this leaflet provided. Consult a health care provider if the symptoms persist after 4 weeks or if they get worse. Before use, consult your doctor if you are diabetic, suffer from reduced blood circulation or recurrent and / or long-term problems with your feet.

7. Shelf life and storage

The shelf life is stated with an expiration date on the packaging. Keep out of the reach and sight of children and at room temperature.

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