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## EN WART PLASTER – FOR FOOT WARTS

Protective plaster with salicylic acid for plantar foot wart. Helps to remove warts and reduces the spread of warts. Convenient one-step application.

### INSTRUCTIONS FOR USE

1. Make sure the skin is clean and dry.
2. Unwrap the plaster and place it on the wart.
3. Change plaster daily. When you remove the plaster, clean the wart with a nail file or a pumice stone to remove dead tissue.
4. Repeat this treatment as needed (until the wart is removed) for up to 12 weeks.

### WARNING

• If you are in any doubt about whether the skin condition is a wart, consult your physician. • Do not use on moles, birthmarks, hairy warts, genital warts, warts on the face, hyperpigmented skin or any other unusual looking skin condition. Do not use on mucous membranes, particularly not the eyes. • Do not use this product on wounds, irritated skin, damaged skin or on any area that is infected or reddened. • If the treated area becomes inflamed, irritated, infected or painful, discontinue use, clean the area with plenty of water and contact your physician. •

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Packaging glue contains latex which may cause allergic reactions

985183-02

Revision date: 03/01/2017

physician or a pharmacist. • Store in a cool and dry place. Ingredients: Polyurethane foam, acrylic adhesive, salicylic acid and azorubine

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Orkla Cederroth S.A.U.