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EN PROTECTIVE CORN PASTER WITH SALICYLIC ACID

Protective plaster with a perforated pad that contributes to pressure relief and thus pain relief. The plaster contains salicylic acid (40%) which helps to eliminate corns and calluses. Our skin is composed of several layers. The outermost layer, epidermis, is constantly regenerated. Its main function is to act as a barrier to protect the deeper layers of the skin. Corns and/or calluses can be formed as a response to repeated pressure or friction (e.g. due to the use of unsuitable footwear).

INSTRUCTIONS FOR USE

1. Make sure the skin is clean and dry.
2. Unwrap the plaster and place it on the corn.
3. Ensure the circle at the center of the plaster is placed directly over the corn or calluses.
4. Change the plaster daily. After four days of treatment it might be possible to eliminate the core of the corn after bathing your foot in hot water containing soap or salt.
5. If necessary, repeat the treatment. Avoid excessive use.

PREVENTION

To prevent re-formation of the corn or callus you need to ensure that the affected area is not exposed to excessive pressure or repeated friction. It is recommended to use a self-adhesive protective ring to corns or calluses. Use comfortable footwear

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Packaging glue contains latex which may cause allergic reactions

0197

985175-03

Revision date: 26/01/2017

WARNINGS

• If any reaction (for example, reddening or smarting) occurs, the product should be removed and not used again. • Do not use if you are hypersensitive to salicylic acid/salicylates. • Do not use this product on wounds, irritated skin, damaged skin or on any area that is infected or reddened. • If you are a diabetic, have poor circulation or renal dysfunction, you have to consult your physician before use. • Do not use on mucous membranes, particularly not the eyes. • Salicylic acid can increase the permeability of the skin, which is important to know if you are using topical medications. • Contact with the healthy skin surrounding the corn should be minimized to prevent irritation. • Not to be used by children under 6 years old or pregnant women, unless recommended by a physician. • The product may only be used once by a single user. Re-use of the plaster increases the risk of infection to yourself or anyone using it. • For external use only. Do not ingest. • Keep out of the reach and sight of children. Store in a cool and dry place. Ingredients: Polyethylene foil, acrylic adhesive, salicylic acid and azorubine.

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For distributors' contact details, please visit
www.salvequick.com

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Orkla Cederroth S.A.U.