Artwork Version:	
Repro Version:	STAGE: 4 VERSION: 1
Job No:	5842493
Cutter Ref:	
Job Title:	Orkla Salvequickmed 51030171 SQuickMed Foot Rescue PL (Leaflet) Export
Design Ref:	DF 405233
Printer:	
Contact:	
Supplier:	Natumin
Contact:	Eva Hollander
SGS Contact:	Louise Blow (P) Sophie Thompson
Operator:	charly.marritt
Modify Date:	08-OCT-19 14:38:17
Design Barcode:	
MySGS Barcode:	NONE
Magnification:	
BWR:	0
Print Process:	Litro
Profile:	ED07 9900SM EO Offset ISO Coated 39L
Substrate:	

Tone - Screen: 150# Angle: 15.0° 0.05% - 47mm²	Tone - Screen: 150# Angle: 45.0° 0.03% - 25mm²
Tone - Screen: 150# Angle: 0.0° 0% - 4mm²	Line - 7.1% - 6937mm²
Line - 100% - 97738mm²	

Total Ink Coverage: 107.62%

Supplier/Printer Sign Off:
Please sign and date against your appropriate title after checking proof and indicate clearly any comment or corrections.

Supplier:	
Date:	
Comments:	
Printer:	
Date:	
Comments:	

Content:	<input type="checkbox"/>	Signed:	<input type="checkbox"/>
Colour:	<input type="checkbox"/>	Signed:	<input type="checkbox"/>
Retoucher Check:	<input type="checkbox"/>	Signed:	<input type="checkbox"/>
		Date:	<input type="checkbox"/>

240 mm

(EN) Instructions for use
Salvequick Foot Rescue
Intended use: Treatment of foot fungus and simultaneous treatment of heel cracks (rhagades), calluses (hyperkeratosis) and dry feet.

Table of Contents

- How does Salvequick Foot Rescue work?
- Ingredients
- Use
- Treatment
- Description of the container
- Warning and precautions
- Shelf life and storage
- Contact information

1. How does Salvequick Foot Rescue work?
Salvequick Foot Rescue is a patented medical device class IIa that has water-binding and antimicrobial properties.

2. Ingredients
Aqua, 1,5-Pentandiol, Cetyl Alcohol, Caprylic/ Capric Triglyceride, Glycerin, Isostearyl Isostearate, Dimethicone, Glyceryl Stearate, PEG-75 Stearate, Ceteth-20, Steareth-20, Xanthan Gum, Artemisia Abrotanum Leaf/Stem Extract, Disodium Phosphate, Sodium Phosphate.

3. Use
Salvequick Foot Rescue can be used to treat and prevent the spread and / or aggravation of foot fungus, heel cracks, calluses and dry feet. To be used by adults. Apply 1 pump on each foot. The volume depends on the size of the treatment area. Apply to clean and dry feet so that the entire area to be treated is covered. Treat both feet to prevent spreading. Wash hands after application to avoid contamination of other areas.

4. Treatment
Apply twice a day until all symptoms are completely gone. It is important not to end the treatment prematurely to prevent re-infection. The treatment time is individual and depends on the severity of the initial symptoms. If no improvement can be seen after 3 weeks, consider alternative treatment and / or contact health care for advice. The same applies if you are diabetic, suffer from reduced blood circulation or recurrent and / or long-term problems with your feet.

5. Description of the container
Salvequick Foot Rescue is packed in an airtight bottle that protects the product from external influences. Thanks to the airtight package, a long shelf life is obtained when the product is protected from the environment. The bottle delivers about 1 ml per pump.

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Hersteller: Natumin Pharma AB Snickaregatan 10 566 33 Habo, Schweden