Artwork Version:	
Repro Version:	STAGE: 2 VERSION: 1
Job No:	5789136
Cutter Ref:	
Job Title:	Orkla Care AB. Salvequickmed Slox Blister Prevention Leaflet (old Foot Care Blister)
Design Ref:	DF 377296
Printer:	
Contact:	
Supplier:	Dilograf
Contact:	Jesus Casbas
SGS Contact:	Louise Blow (P)
Operator:	adrian.halls
Modify Date:	04-JUL-19 11:29:52
Design Barcode:	
MySGS Barcode:	TBC
Magnification:	
BWR:	0
Print Process:	Litho
Profile:	EC07 9800SM ECI Offset ISO Coated 33L
Substrate:	

PANTONE 2757 C

Line -

6.23% - 2655mm<sup>2</sup>

Total Ink Coverage: 6.88%

#### Supplier/Printer Sign Off:

Please sign and date against your appropriate title after checking proof and indicate clearly any comment or corrections.

Supplier:	
Date:	
Comments:	
Printer:	
Date:	
Comments:	

INTERNAL USE ONLY APPROVED FOR DESPATCH	
Content:	<input type="checkbox"/> Signed:
Colour:	<input type="checkbox"/> Signed:
Retoucher Check:	<input type="checkbox"/> Signed:
	Date:

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**EN Blister Prevention** blister plasters are made from an advanced hydrocolloid material. The plaster absorbs moisture from the skin and creates a supportive cushion that provides instant relief from pressure and friction. Research shows that a moist environment is beneficial for wound healing. The moist environment helps the wound to heal faster and reduces the risk of scars. The plaster is breathable and protects against dirt.

#### USER INSTRUCTIONS:

1. The skin should be completely clean, dry and free from any lotion.
2. Remove the printed protective paper.
3. Loosen one half of the plastic release liner along the perforated line.
4. Remove the plaster from the plastic release liner and apply the plaster centered over the blister.
5. Keep the plaster on until it falls off naturally. Grabbing one end and stretching carefully along the skin easily removes the plaster.

#### WARNING:

- Do not use on blood blisters or deep, heavily bleeding or infected blisters.
- People with impaired wound healing, e.g. diabetics/elderly people should check their blister regularly.
- Consult a doctor if an infection occurs or if the blister has not healed within a week.
- Product to be used only once by one user. Re-use of the product increases the risk of infection.

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