Artwork Version:	
Repro Version:	STAGE: 8 VERSION: 1
Job No:	5791668
Cutter Ref:	
Job Title:	Orkla Care AB. Salvequickmed Salvequick Blister Prevention Exp (Leaflet)
Design Ref:	DF 377297
Printer:	
Contact:	
Supplier:	Dilograf
Contact:	Jesus Casbas
SGS Contact:	Louise Blow (P) Sophie Thompson
Operator:	adrian.halls
Modify Date:	25-OCT-19 10:34:31
Design Barcode:	
MySGS Barcode:	NONE
Magnification:	
BWR:	0
Print Process:	Litho
Profile:	EC07 9900SM ECI Offset ISO Coated 39L
Substrate:	

PANTONE 2757 C 

Line -

10.51% - 4481mm<sup>2</sup>

Total Ink Coverage: 11.17%

#### Supplier/Printer Sign Off:

Please sign and date against your appropriate title after checking proof and indicate clearly any comment or corrections.

Supplier:	
Date:	
Comments:	
Printer:	
Date:	
Comments:	

Content:	<input type="checkbox"/>	Signed:	
Colour:	<input type="checkbox"/>	Signed:	
Retoucher Check:	<input type="checkbox"/>	Signed:	
		Date:	

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**EN Blister Prevention** blister plasters are made from an advanced hydrocolloid material. The plaster absorbs moisture from the skin and creates a supportive cushion that provides instant relief from pressure and friction. Research shows that a moist environment is beneficial for wound healing. The moist environment helps the wound to heal faster and reduces the risk of scars. The plaster is breathable and protects against dirt.

#### USER INSTRUCTIONS:

- The skin should be completely clean, dry and free from any lotion.
- Remove the printed protective paper.
- Loose one half of the plastic release liner along the perforated line.
- Remove the plaster from the plastic release liner and apply the plaster centered over the blister.
- Keep the plaster on until it falls off naturally. Grabbing one end and stretching carefully along the skin easily removes the plaster.

#### WARNING:

- Do not use on blood blisters or deep, heavily bleeding or infected blisters.
- People with impaired wound healing, e.g. diabetics/elderly people should check their blister regularly.
- Consult a doctor if an infection occurs or if the blister has not healed within a week.
- Product to be used only once by one user.
- Re-use of the product increases the risk of infection.

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