 <p><b>COLLABRA</b> ARTWORK MANAGEMENT</p> <p><b>PMgr:</b> Joanna Grånström <b>Producer:</b> Erika Feldt</p>	 <p><b>Customer:</b> Orkla Care <b>Contact:</b> Mikaela Sköldbberg</p>	<p><b>Project code:</b> CED6439</p> <p><b>Revision:</b> 1 <b>Date:</b> 2020-03-31</p> <p><b>Salvequick Hydrogel Leaflet</b></p> <p><b>Ref no:</b> - <b>Material no:</b> 985228-01 <b>Old mat no:</b> 985217-02 <b>In. EAN13:</b> - <b>Out. EAN13:</b> -</p> <p><b>Line drawing:</b> - <b>Pack:</b> - <b>Printer:</b> Dilograf <b>Print:</b> Offset</p>	<p><b>INFO FOR REPRO/PRINTER</b></p> <p>This artwork may NOT be changed or adjusted without prior notice to Collabra. The graphics may NOT be printed without written approval from Collabra or Orkla.</p> <p><b>DIGITAL PROOF (PDF)</b> to <a href="mailto:proof@collabra.se">proof@collabra.se</a></p> <p><b>COLOR PROOF</b> to Orkla AB Mikaela Sköldbberg Box 1336 SE-171 26 Solna Sweden</p>
		<p>FONTS WILL NOT BE SUPPLIED WITH THIS ARTWORK; ALL TEXTS ARE VECTORIZED. ALL SPECIAL COLOURS SPECIFIED ARE FROM THE PANTONE® MATCHING SYSTEM UNLESS OTHERWISE STATED. THE LINKED IMAGES MAY BE LOW RESOLUTION FOR DECREASING FILE SIZE PURPOSES AND SHOULD BE REPLACED BY HIGH RESOLUTION IMAGES BEFORE PRINTING. THE COLOURS ON THIS PRINTOUT ARE NOT ACCURATE AND ARE INTENDED TO BE USED AS A GUIDE ONLY; DO NOT USE FOR MATCHING PURPOSES UNLESS OTHERWISE STATED. THIS ARTWORK DOES NOT CONTAIN ANY TRAP/GRIP/CHOKO INFORMATION.</p>	

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**BLISTER RESCUE**  
360° Protective care

Orkla Care AB (Svetsarvägen 15),  
Box 1336, SE-171 26 Solna, Sweden

For distributors' contact details, please visit [www.salvequick.com](http://www.salvequick.com)

**EN Blister Rescue** is intended for use on punctured or unpunctured blisters on feet, and also for blister prevention. The product provides a complete 360° Protective Care of the blister, with the following five functions:

- Protects blistered skin.
- Easy to apply.
- Stays in place securely.
- Easy and painfree to remove.
- Discreet, transparent and adapts to skin colour.

**INSTRUCTIONS:**

1. Clean and dry the skin from dirt and lotion before applying the plaster. Tear to open the protective plaster pouch.
2. Remove the plaster's protective paper (marked 1).
3. Apply the plaster centered over the blister and try to minimize wrinkles. Remove the white folded tab.
4. Remove the protective plastic top layer (marked 2).

You can keep the plaster on until it falls off, not more than five days. Change if the gel pad gets saturated.

**WARNING:**

- Do not use on blood blisters or deep, heavily bleeding or infected blisters.
- People with impaired wound healing, e.g. diabetics/elderly people, should check their blister regularly.

- Consult a doctor if an infection occurs or if the blister has not healed within a week.
- Product to be used only once by one user. Re-use of the product increases the risk of infection.

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