**Distributor** Indicates the entity distributing the medical device into the locale. **Dystrybutor** Wskazuje na dystrybutora wyrobu medycznego w danym regionie.

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**Country of manufacture. Kraj wytwarzania.**

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**EC REP**  
 E C Rep Ltd,  
 5 Fitzwilliam Square East,  
 Dublin 2, D02 R044, Ireland.  
 Tel: +353 1 2 544 941  
 Email: info@ecrep.ie

**Pharmaplast S.A.E**  
 Amria free zone 23512, Alexandria, Egypt.  
[www.pharmaplast-online.com](http://www.pharmaplast-online.com)

#### EN PROTECTIVE CORN PLASTER WITH SALICYLIC ACID

Protective plaster with a perforated pad that contributes to pressure relief and thus pain relief.

The plaster contains salicylic acid (40%) which helps to eliminate corns and calluses. Our skin is composed of several layers. The outermost layer, epidermis, is constantly regenerated. Its main function is to act as a barrier to protect the deeper layers of the skin. Corns and/or calluses can be formed as a response to repeated pressure or friction (e.g. due to the use of unsuitable footwear).

Salvequick Corn plasters: Helps remove corns.

- FEATURES:**
- Salicylic acid discs target corn & callus area without affecting surrounding skin.
  - Helps remove corns & callus without pain.
  - Fast and effective.
  - Cushions rings reduce pressure on corn, thus helps relieve pain immediately.

- INSTRUCTIONS FOR USE:**
1. Soak the corn in warm water for 5 minutes before application.
  2. Clean and dry skin before use, ensuring targeted area is free of creams and oils.
  3. Remove the release liner, avoiding touching the adhesive side.
  4. Apply plaster with the embedded salicylic

- acid disc centrally over the corn, ensuring the edges are well smoothed down.
5. Leave in place until it starts to detach. After maximum 48 hours, remove the plaster.
6. Soak the affected area in warm water for 5 minutes before removing loosened skin.
7. Repeat this procedure as necessary for up to 14 days.

**Contraindication:**  
 Do not use if you are diabetic or hypersensitive to salicylic acid or suffer from poor blood circulation.  
 Pregnant women and children: only use with doctor approval.

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